

# TEOSYAL



EFFICACY • SAFETY • DURABILITY

Teosyal® , a complete range of anti-ageing injectable hyaluronic acid based products.

With Teosyal®, Teoxane Laboratories have developed and created a completely new range of anti-ageing products, which enables doctors to treat wrinkles, cutaneous depressions, to correct facial contour defects and also to revitalise the dermis under the best conditions.

The Teosyal® range consists of five monophasic hyaluronic acid based products of non-animal origin, which are highly biocompatible, offering patients immediate and long-lasting results with no preliminary testing.

### Technique of injection

The treatment session lasts about 30 minutes. For the comfort of the patient and depending on the zone to be treated, a local anesthetic or anesthetizing cream may be applied.

Teosyal 27G and 30G are injected into the mid to deep dermis using linear threading injection technique: introduce the needle along the length of the cutaneous depression then slowly introduce the product whilst withdrawing the needle. Thoroughly massage the injected sites in order to ensure a uniform distribution of the product. Depending upon the extent of the correction sites, a further application after six to eight weeks may be useful.

Teosyal Meso is injected uniformly and superficially over the treatment sites using the multipuncture technique. Manually stretch the treatment sites and carry out multipunctures of two to three millimetres in diameter separated by one centimetre to a depth of approximately two millimetres. The injection volume depends upon the extent of the correction sites.



36 year old female patient 1 month after injection of Teosyal 27g to nasal labial lines.



TELEPHONE: 0844 800 9631  
[www.celllife.co.uk](http://www.celllife.co.uk)

Company reg. 6152024